

Kappa/Lambda Light Chains - Urine

Order Name: **LIGHT CH U**

Test Number: 2051750

Revision Date: 01/01/2022

TEST NAME	METHODOLOGY	LOINC CODE
Kappa Light Chains Urine	Nephelometry	25682-6
Measured Kappa Chain	Nephelometry	15188-6
Lambda Light Chains Urine	Nephelometry	25684-2
Measured Lambda Chain	Nephelometry	15187-8
Comment Kappa Lambda	Nephelometry	
Total Volume - Prompt	Prompt	
Collection Duration	Prompt	n/a

SPECIMEN REQUIREMENTS				
Specimen	Specimen Volume (min)	Specimen Type	Specimen Container	Transport Environment
Preferred	2mL (0.5mL)	Urine, 24-hour	24 hour Urine Container	Refrigerated
Instructions	Collect without preservative. Record total volume on 24hr Container and all aliquots. Specimen Stability: Room Temperature: 12 hours, Refrigerated: 7 days, Frozen Not Established			

GENERAL INFORMATION	
Testing Schedule	Mon-Sat
Expected TAT	1-3 days
Notes	Reference Lab: Quest Test Code: 17300 Click Here to view information on the Quest website. Note test is for 24hr Urine Collections. For Random Urine Collection please send in a Miscellaneous order for Quest 34318 Kappa Lambda Light Chains, Total, Random Urine
CPT Code(s)	83521X2
Lab Section	Reference Lab