

Uric Acid Urine Timed

Order Name: **URIC TM U**

Test Number: 3006175

Revision Date: 06/04/2021

TEST NAME	METHODOLOGY	LOINC CODE
Total Urine Volume		3167-4
Creatinine, Urine, mg/dL	Kinetic Alkaline Picrate (Jaffe)	20624-3
Creatinine Urine Timed Estimated		
Creatinine, Urine, 24 Hour		2162-6
Uric Acid 24 Hour Urine mg/dL	Uricase	21587-1
Uric Acid Urine Timed Estimated		
Uric Acid 24 Hour Urine mg/24hr		3087-4
Hours Collected - Prompt		30211-7
Volume 24 Urine Prompt		

SPECIMEN REQUIREMENTS				
Specimen	Specimen Volume (min)	Specimen Type	Specimen Container	Transport Environment
Preferred	1 mL (0.5)	Urine, 24-hour	24 hour Urine Container	Refrigerated
Instructions	Timed urine collection. No preservative. Record number of hours and volume in ml on the specimen container. Keep refrigerated. Specimen stability: Ambient 24 hours (4 days if pH adjusted >8.0 dropwise with NaOH). Refrigerated 7 days.			

GENERAL INFORMATION	
Testing Schedule	Daily
Expected TAT	1-2 days
Clinical Use	Useful for assessment and management of patients with kidney stones and gout.
CPT Code(s)	84560 (add 81050 if urine measured by RML)
Lab Section	Chemistry