Labcorp Oklahoma, Inc. Test Directory

Soluble Transferrin Receptor

Order Name: SOLUBL TRN

Test Number: 3604710 Revision Date: 11/20/2024

TEST NAME	METHODOLOGY	LOINC CODE
Soluble Transferrin Pecenter	Immunochemiluminometric (ICMA)	

Soluble Transferrin Receptor Immunochemiluminometric (

SPECIMEN REQUIREMENTS				
Specimen	Specimen Volume (min)	Specimen Type	Specimen Container	Transport Environment
Preferred	0.5 mL (0.2 mL)	Serum	Clot Activator (Red Top, No-Gel)	Refrigerated
Alternate 1	0.5 mL (0.2 mL)	Plasma	EDTA (Lavender Top)	Refrigerated
Alternate 2	0.5 mL (0.2 mL)	Plasma	Sodium Heparin (Green Top / No- Gel)	Refrigerated
Instructions	Notes: 0.2 mL (Note: This volume Does NOT allow for repeat testing.) Specimen Type: Red-top tube or green-top (heparin) tube or lavender-top (EDTA) tube Specimen Storage: Refrigerated Specimen Collection: Separate serum or plasma from cells and transfer to a plastic transport tube. Specimen Stability: Ambient: 14 days, Refrigerated: 14 days, Frozen: 14 days			

GENERAL INFORMATION		
Expected TAT	2-3 Days	
Clinical Use	Elevated sTfR values are found in anemia of chronic disease (ACD), iron deficiency anemia (IDA), polycythemia, hemolytic anemia, thalassemia, hereditary spherocytosis, sickle cell and megaloblastic anemia, myelodysplastic syndrome and vitamin B12 deficiency. Elevated sTfR concentrations occur during pregnancy when there is a deficiency of functional iron.	
Performing Labcorp Test Code	143305	
CPT Code(s)	84238	
Lab Section	Reference Lab	

Service provided by Labcorp Oklahoma, Inc. All Rights Reserved. © 2003 - 2025