

## Vaginosis Profile Saline

Order Name: **VAG PROF**  
Test Number: 2915425  
Revision Date: 09/17/2018

| TEST NAME                           | METHODOLOGY | LOINC CODE |
|-------------------------------------|-------------|------------|
| Ph of vaginal discharge             |             | 48037-6    |
| Whiff test                          | Amine       | 46730-8    |
| Gram stain                          |             |            |
| Trichomonas Microscopic Examination | Microscopy  | 32766-8    |
| Vaginal Yeast Examination           | Microscopy  | 32765-0    |
| Clue Cell Examination               | Microscopy  | 32764-3    |

### SPECIMEN REQUIREMENTS

| Specimen            | Specimen Volume (min)   | Specimen Type | Specimen Container | Transport Environment |
|---------------------|---|---------------|--------------------|-----------------------|
| Preferred           | See Instructions  | Swab          | Sterile Saline     | Room Temperature      |
| <b>Instructions</b> | <p>Obtain specimen from mucosal surface and place swab in saline tube.<br/>Samples contaminated with preparations containing iodine or by the immediate prior use of vaginal lubricants are not recommended.<br/>For optimal results, testing should be performed as soon as possible following collection; specimens must be &lt;24 hours old upon arrival.</p> <p>Note: The Vaginal Profile from Swab (with Trichomonas Antigen), is recommended for specimens being testing &gt;2 hours after collection. Please refer to test 2915445 for collection information.</p> |               |                    |                       |

### GENERAL INFORMATION

|                  |   |
|------------------|---|
| Testing Schedule | Daily   |
| Expected TAT     | 1 Day   |
| Clinical Use     | Determines bacterial vaginosis or yeast vaginitis |
| CPT Code(s)      | 83986; 82120; 87210; 87205                        |
| Lab Section      | Microbiology                                      |