

## Immunoglobulin IgG, IgA, IgM Quantitative

Order Name: **IG G-A-M**  
Test Number: 5001090  
Revision Date: 11/13/2018

TEST NAME	METHODOLOGY	LOINC CODE
Immunoglobulin IgG Quantitative	Turbidometric	2465-3
Immunoglobulin IgA Quantitative	Turbidometric	2458-8
Immunoglobulin IgM Quantitative	Nephelometry	2472-9

### SPECIMEN REQUIREMENTS

Specimen	Specimen Volume (mL)	Specimen Type	Specimen Container	Transport Environment
Preferred	2mL (1mL)	Serum	Clot Activator SST	Refrigerated
Instructions	Allow specimen to clot completely at room temperature. Separate serum from cells ASAP or within 2 hours of collection. Stability After separation from cells: Ambient 4hours, Refrigerated 7 days, Frozen 1 months (avoid repeated freeze/thaw cycles).			

### GENERAL INFORMATION

Testing Schedule	Mon - Sat
Expected TAT	1-3 Days
CPT Code(s)	82784x3
Lab Section	Immunology - Serology