

## Trichomonas Urine - TMA

Order Name: **Trich Urine**

Test Number: 6910123

Revision Date: 07/11/2017

TEST NAME	METHODOLOGY	LOINC CODE
Trichomonas Urine	Transcription-Mediated Amplification	70167-2

### SPECIMEN REQUIREMENTS

Specimen	Specimen Volume (min)	Specimen Type	Specimen Container	Transport Environment
Preferred	2 mL	Urine, First Void	APTIMA Urine Transport Tube	Refrigerated

**Instructions**  
 First catch urine. Patient should not have urinated for at least 1 hr prior to collection. Female patients should not cleanse the labial area prior to collection. Transfer 2 mL urine to APTIMA Urine Specimen Transport Tube within 24hrs of collection. Liquid level must be between fill lines on tube. Stability: Refrigerated = 30 days.

**Please contact your account rep if training and APTIMA Urine Specimen Transport supplies have not been supplied to your practice.**

### GENERAL INFORMATION

Testing Schedule	Mon-Fri
Expected TAT	1-2 days after set-up
CPT Code(s)	87661
Lab Section	Molecular Diagnostic