Labcorp Oklahoma, Inc. Test Directory

Iodine Serum

Order Name: **lodine Serum**Test Number: 5613569
Revision Date: 12/12/2022

TEST NAME			METHODOLOGY		LOINC CODE
lodine Serum			Quantitative Inductively Coupled Plasma-Mass	Spectrometry	2494-3
SPECIMEN REQUIREMENTS					
Specimen	Specimen Volume (min)	Specimen Type	Specimen Container	Transport En	vironment
Preferred	1 mL (0.4 mL)	Plasma	Royal Blue - EDTA (Trace Element Free)	Room Temp	erature
Alternate 1	1 mL (0.4 mL)	Serum	Royal Blue - No Additive Clot (Trace Element Free)	Room Temp	erature
Instructions	Specimen Type: Royal blue-top (EDTA) tube OR royal blue-top without EDTA Specimen Storage: Maintain specimen at room temperature. Specimen Collection: Separate serum or plasma immediately after the collection, and transfer to a certified metal-free transport tube (PeopleSoft No. 111166) for shipment to the laboratory. Specimen Stability: Ambient: 14 days, Refrigerated: 14 days, Frozen: 14 days				

GENERAL INFORMATION		
Expected TAT	2 - 4 days working day.	
Clinical Use	Recommended for determination of iodine excess and monitoring iodine overload in patients administered iodine-containing medications. This test reports total iodine from all iodine-containing species present in the specimen but does not determine the chemical form (species) of the iodine present. For the assessment of nutritional status, Iodine, Urine is preferred.	
Performing Labcorp Test Code	070034	
Notes	Labcorp Test Code: 070034	
CPT Code(s)	83018	
Lab Section	Reference Lab	

Service provided by Labcorp Oklahoma, Inc. All Rights Reserved. © 2003 - 2025