

## HemePath Consult with Client Submitted CBC

Order Name: **PBS Client**  
 Test Number: 2904690  
 Revision Date: 08/02/2024

TEST NAME	METHODOLOGY	LOINC CODE
HemePath Consult with Client Submitted CBC		

### SPECIMEN REQUIREMENTS

Specimen	Specimen Volume (min)	Specimen Type	Specimen Container	Transport Environment
Preferred	See Instructions	Peripheral Blood	Clipped Corner Glass Slides with Holder	Room Temperature

**Instructions**

Please submit a copy of the CBC results performed in your lab, patient history, and two (2) unstained blood smears. Smears should be made on 26 x 75mm, white, 45-degree **clipped corner slides**.

The peripheral blood smear, laboratory data and clinical history will be reviewed by a pathologist who will issue an interpretative report.  
 Note: We cannot guarantee a review will be performed if smears are submitted on **non-clipped corner slides**.  
**Reference Memorandum from Nov. 1, 2023**

Pathology Report will be issued on this on Order/Result Code: [8077777]

### GENERAL INFORMATION

Testing Schedule	See Notes
Expected TAT	1-3 Days
Clinical Use	Client submitted CBC with automated differential and peripheral blood smear. The pathologist consult only will be billed.
Notes	<p>For clients requesting Labcorp Oklahoma, Inc. Central Lab to perform the CBC and blood smear in addition to the pathologist review, Please continue to order the standard <b>HemePath Consult (2904600)</b> or for anemia evaluation, order <b>HemePath Consult Analyzer (0110800)</b>.</p> <p>Please note, an EDTA lavender tube must be received in the lab within 24 hours of collection.</p> <p><b>Testing Schedule:</b>  <b>Monday, Wednesday, and Friday</b> if <b>non-clipped corner slides</b> are sent.  <b>Monday through Friday</b> if <b>clipped corner slides</b> are sent.</p>
CPT Code(s)	Code determined with clinical findings
Lab Section	Pathology