Porphyrins, Feces

Order Name:Porph FecesTest Number:5194403Revision Date:01/04/2025

TEST NAME		ME	THODOLOGY	LOINC CODE	
Porphyrins, Feces	High Performance Liquid Chromatography				
SPECIMEN REQUIRE	EMENTS				
Specimen	Specimen Volume (min)	Specimen Type	Specimen Container	Transport Environment	
Preferred	100g	Timed Stool Collection	Sterile Screwtop Container	Frozen	
Instructions	Patient Preparation: Patient should refrain from ingesting red meat and aspirin-containing products (eg, Excedrin, Aspirin) for 3 days prior to specimen collection.				
	Specimen: Entire collection (48, 72, or 96 hour) (min 10g) Stool in Sterile container Frozen,				
	Note: 24-Hour collection is adequate if the collection volume is approximately 100 g. Length of collection period is required.				
	Specimens smaller than 100 g may not provide interpretable results. Include a list of medications the patient is currently taking.				
	Stability Requirements: Room temperature n/a, Refrigerated 14 days, Frozen 14 days				
	Cause for Rejection: Specimen less than 10 grams.				
	Special Instructions: Testing referred to Mayo Medical Lab TC#FQPPS				
GENERAL INFORMA	TION				

GENERAL INFORMATION		
Expected TAT	13 - 18 days	
Performing Labcorp Test Code	820730	
CPT Code(s)	84126	
Lab Section	Reference Lab	