

Parasite Screen - Stool (Giardia and Cryptosporidium Antigen)

Order Name: **C PARA SC**  
Test Number: 6060300  
Revision Date: 07/22/2022

TEST NAME	METHODOLOGY	LOINC CODE
Parasite Screen - Stool (Giardia and Cryptosporidium Antigen)	Enzyme Immunoassay	

SPECIMEN REQUIREMENTS				
Specimen	Specimen Volume (min)	Specimen Type	Specimen Container	Transport Environment
Preferred	3 grams	Stool, Random	Formalin and PVA container	Room Temperature
Instructions	<p><b>Stool in formalin screwtop container.</b></p> <p>Formalin can be added to fresh (undiluted), refrigerated specimens up to 72 hours old; formalin can be added to room temperature fresh (undiluted) specimens up to 8 hours old. Testing should be performed with in 1 month.</p> <p><b>Rejection Criteria:</b></p> <ul style="list-style-type: none"><li>Specimens received in grossly leaking containers.</li><li>Specimens in Eco-Fix transport media.</li><li>Specimens received in commercially prepared transport vials that are overfilled or underfilled with the patient specimen will be rejected for testing.</li><li>Colonoscopy and colon wash specimens cannot be tested.</li><li>Raw stool specimens received in the laboratory more than 2 hours after collection cannot be tested.</li></ul>			

GENERAL INFORMATION	
Testing Schedule	Sun-Sat
Expected TAT	1-3 Days
Clinical Use	Testing includes specific antigen testing for for Giardia lamblia and Cryptosporidium sp.
Notes	Refer to the Microbiology page in the Specimen Collection section of our service guide for more information on Stool Collection Containers. <b>Please make sure your order and result code is 6060300 and not the old 6060200.</b>
CPT Code(s)	87328, 87329
Lab Section	Microbiology