Labcorp Oklahoma, Inc. Test Directory

Immunoglobulin IgM Quantitative

Order Name: IGM

Test Number: 5001200
Revision Date: 10/03/2017

TEST NAME			METHODOLOGY	LOINC CODE
Immunoglobulin IgM Quantitative			Nephelometry	2472-9
SPECIMEN REQU	JIREMENTS			
Specimen	Specimen Volume (min)	Specimen Type	Specimen Container	Transport Environment
Preferred	1 mL	Serum	Clot Activator SST	Refrigerated
Instructions	·	Allow specimen to clot completely at room temperature. Separate serum or plasma from cells ASAP or within 2 hours of collection. Stability After separation from cells: Ambient 4hours, Refrigerated 7 days, Frozen 1 months (avoid repeated freeze/thaw cycles).		

GENERAL INFORMATION		
Testing Schedule	Mon - Sat	
Expected TAT	3 Days	
Clinical Use	Useful for the detection of monoclonal gammopathies and immune deficiencies.	
CPT Code(s)	82784	
Lab Section	Immunology - Serology	

Service provided by Labcorp Oklahoma, Inc. All Rights Reserved. © 2003 - 2025