

Helicobacter pylori Antigen Detection Stool

Order Name: **HPYLORI AG**  
Test Number: 3502325  
Revision Date: 09/21/2022

| TEST NAME                                   | METHODOLOGY      | LOINC CODE |
|---|------------------|------------|
| Helicobacter pylori Antigen Detection Stool | Immunoassay (IA) | 80373-4    |

| SPECIMEN REQUIREMENTS |  |               |                          |                       |
|-----------------------|--|---------------|--------------------------|-----------------------|
| Specimen              | Specimen Volume (min)  | Specimen Type | Specimen Container       | Transport Environment |
| Preferred             | 5g (1)   | Stool, Random | Stool specimen container | Refrigerated          |
| Instructions          | <p><b>Patient Preparation:</b> To avoid false negative results, no antibiotics, proton pump inhibitors, or a bismuth preparation should be administered to patient for 14 days prior to specimen collection.</p> <p>Transfer 5g(1g) stool to an unpreserved stool transport vial and Freeze.</p> <p><b>Unacceptable Conditions:</b> Gastric specimens, swabs or tissue. Specimens in media or preservatives.</p> <p><b>Stability:</b> Ambient: 2 hours; Refrigerated: 72 hours; Frozen: 1 month.</p> |               |                          |                       |

| GENERAL INFORMATION |              |
|---------------------|--------------|
| Testing Schedule    | Sun-Sat      |
| Expected TAT        | 2 Days       |
| CPT Code(s)         | 87338        |
| Lab Section         | Microbiology |