

MySwab Vaginosis Profile Swab - TMA

Order Name: **MySwab Vag**
 Test Number: 6987003
 Revision Date: 04/06/2020

TEST NAME	METHODOLOGY	LOINC CODE
Bacterial Vaginosis - Swab	Transcription-Mediated Amplification	46999-9
Candida Species - Swab	Transcription-Mediated Amplification	47000-5
Candida Glabrata - Swab	Transcription-Mediated Amplification	n/a

SPECIMEN REQUIREMENTS				
Specimen	Specimen Volume (min)	Specimen Type	Specimen Container	Transport Environment
Preferred	Swab	Swab	APTIMA Multitest Swab kit	Refrigerated
Instructions	Collect Swab in APTIMA Multitest Swab collection kit. Place swab in Swab Specimen Transport Tube, break shaft off at scoreline then recap tube. Stability: Room Temperature 30 Days, Refrigerated 30 Days, Frozen: N/A			

GENERAL INFORMATION	
Testing Schedule	Mon-Fri
Expected TAT	1-2 days after set-up
CPT Code(s)	87801, 87481
Lab Section	Molecular Diagnostic